

Tips for Crowns under Partials

Crown under Partial Techniques

A patient who needs a crown placed under a partial denture is one of the more intricate impressions required in order to get a proper fitting prosthetic. This task can be accomplished by following the tips suggested below.

Partial Inclusion Technique

Whenever possible it's best to pull the partial in a PVC impression with the prep exposed and send it to the lab for fabrication.

Please send a full arch impression to ensure optimum fit and seat for your final restoration.

When using this method please make sure the partial is fully seated before taking the impression.

Using this method will provide the technician with everything needed to fabricate a prosthetic that seats correctly.

Scheduling Your Patient

Also we are able to deliver the partial and the finished crown within 5 working days. This includes three full working days, one day for pickup and one day for delivery. You may also schedule your patient in advance so that way the patient is not inconvenienced with multiple pickups and/or appointments.

Please feel free to call the lab direct with any questions or more options that are available regarding this type of restorative service.